Schwenkschieber Extendable Arm

DE

EN

Montagehinweis
Assembly Instructions

CE

9000-606-20/30

 DÜRR
DENTAL

2012/10

Montagehinweis Schwenkschieber

Best. Nr. 7600G150-00

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Assembly Instructions Extendable Arm


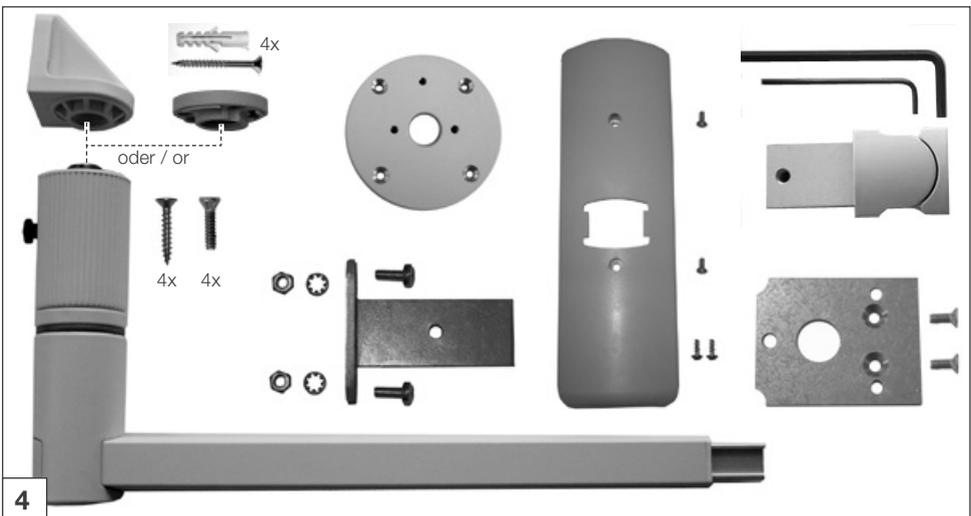
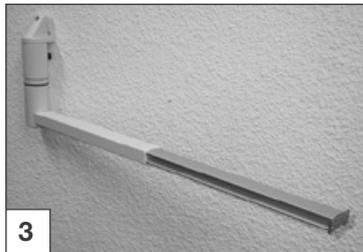
Order number 7600-150-00


Fig. 1:
Extendable arm with hose holder
assembly Comfort
(For extendable arm with Dürre hose
holder assembly Eco, see section 4
below)

Fig. 2:
Extendable, c. 70 cm

Fig. 3:
Rotational 360 °

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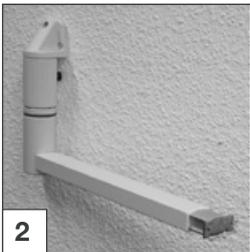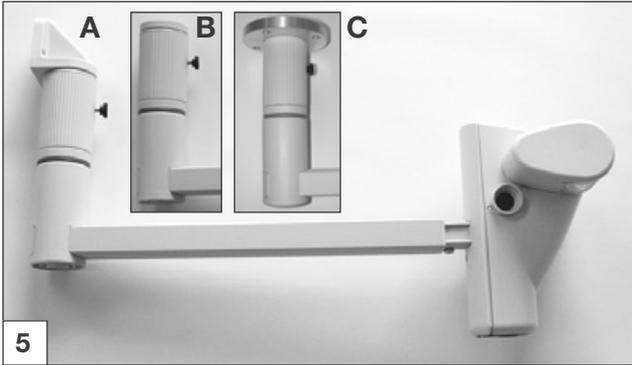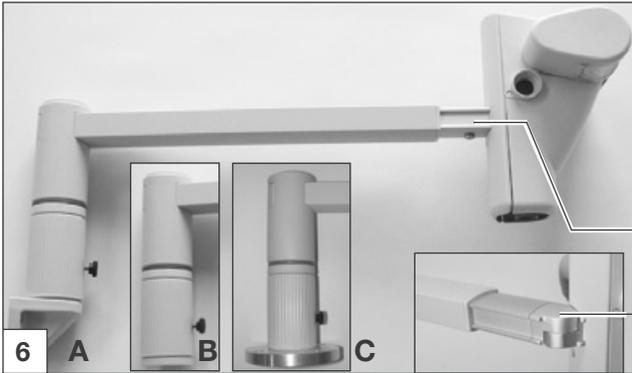
2. Installation alternatives

The Dürr extendable arm can be supplied with (2) or without (1) rotational element, as required.

- Wall mounting with part (A) or (A1)
- Mounting below or on working surface with part (B) or (C)

The rotational parts of the extension arm can be variably fitted together, depending on the amount of space available.

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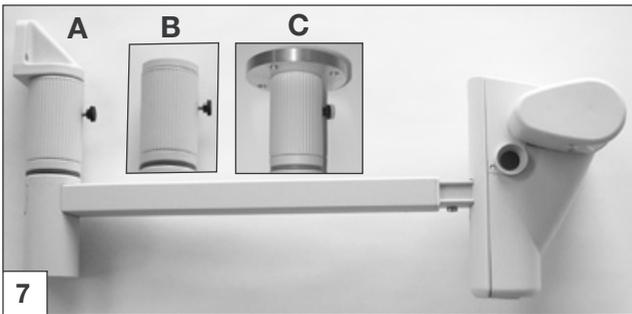
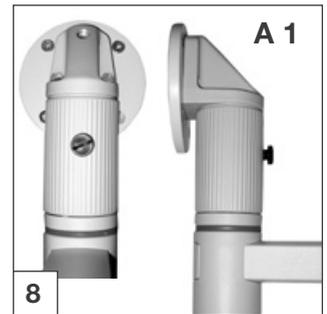
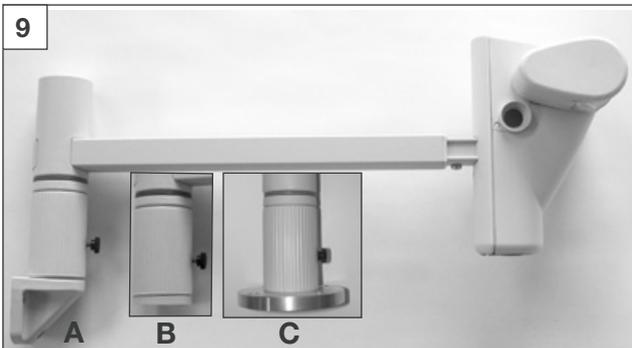
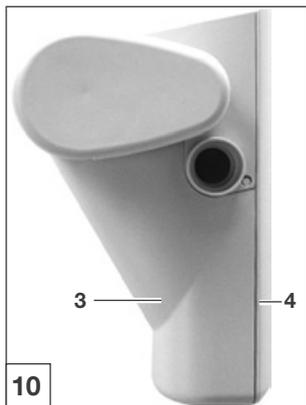
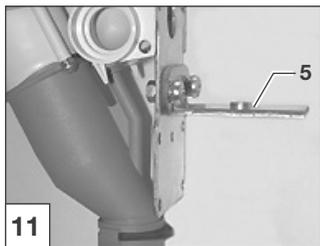
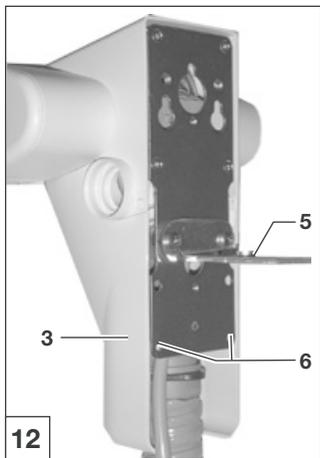
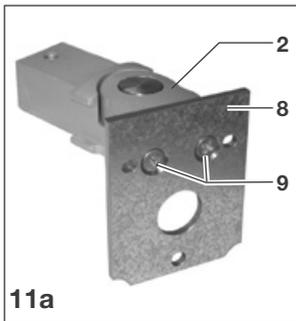
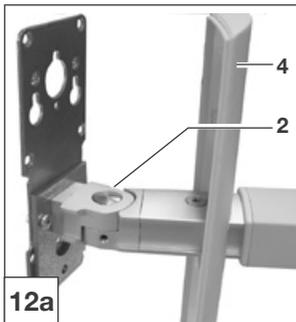
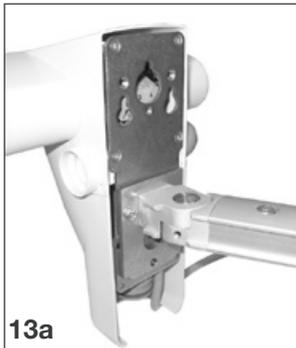
3. Installation Extendable arm on hose holder assembly Comfort

Fixing the hose manifold to the extendable arm can be carried out either using an angle bracket (5) or a rotational element (2).

- Remove the rear cover (4) and the front cover (3).

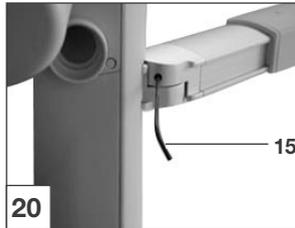
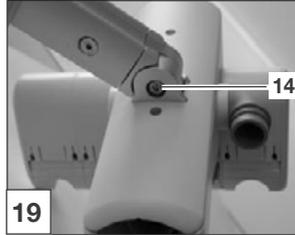
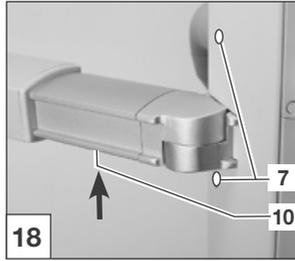
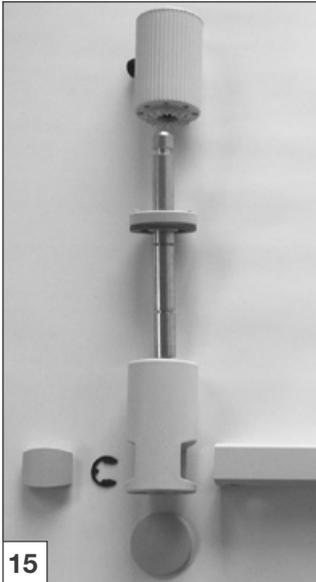
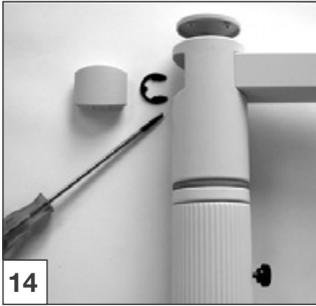
3.1 Initial installation to manifold

Fixing with angle bracket (5)

- Mount the angle bracket (5) using 2 screws, spring washers and nut onto the mounting plate, Fig. 11.
- Place drainage hose in position and tighten.
- Replace cover (3) and fix using the 2 screws (6).
- Place the rear cover in position (Fig. 13) and tighten with screws M3 (7).

Fixing with rotational element (2)

- Screw the adapter (8) onto the rotational element (2) using the two screws (9) supplied (fig 11a).
- Screw the rotational element (2) onto the back of the manifold using the screws and washers supplied (fig 12a+13a).
- Slide the rear cover (4) onto the extendable arm.

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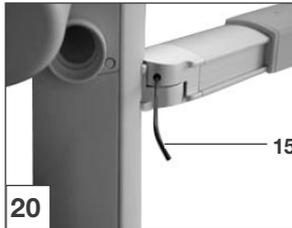
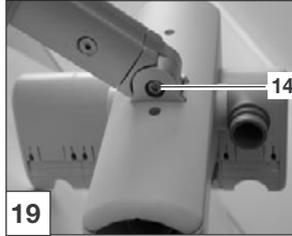
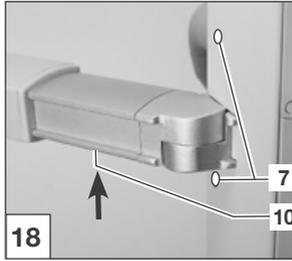
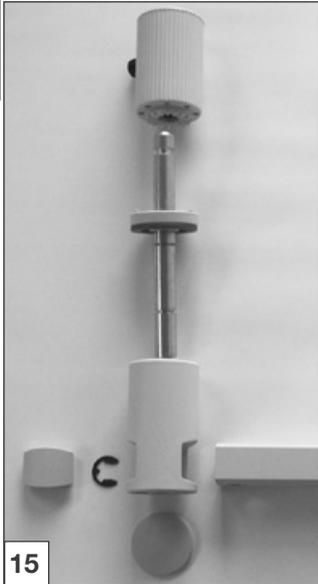
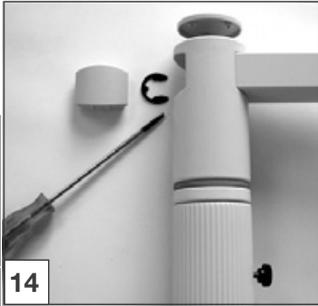
3.2 Preparing the extendable arm

Before fixing the extendable arm to the hose holder assembly, check which assembly position is most suitable for the surgery, see section 2 above.

- In order to change parts, a screwdriver can be used (Fig. 14) to remove the cover and the securing clip, and the parts disassembled (Fig. 15)
- Assemble the parts, as desired, as shown in the alternative assembly examples.
- Secure with the clip and replace the cover

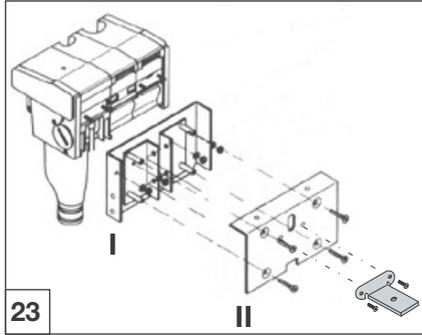
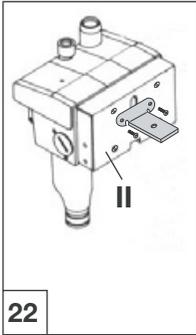
3.3 Mounting hose holder onto extendable arm

Angle bracket (Figs 16+17)

Place the angle bracket (5) into the rail of the extendable arm (11) and secure in place with the screw (12).

Rotational element (Figs 18+19)

- Place the rotational element into the extendable arm and tighten with the screw (10).
- Replace covers (3 and 4; see Fig. 10) and tighten the rear cover in place using the screws (7) provided.
- The moving of the rotational element can be adjusted by using the Allen key provided to tighten and loosen the socket screw (14) as shown in Fig. 19. Fine adjustment can be carried out with the Allen key on socket screw (15) as seen in Fig. 20.

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4. Mounting the extendable arm onto an Dürre hose holder Eco/Selektiv

- Mount angle bracket (5) on part II of mounting plate. (Fig. 22)
- On some models it may be necessary to mount part I onto the hose holder before the angle bracket (5) is fixed onto the plate, II.
- Secure part II to part I with screws
- (Fig. 23).
- Now proceed as under section 3.2 or 3.3 above

DÜRR DENTAL AG
Höpfigheimer Strasse 17
74321 Bietigheim-Bissingen
Germany
Fon: +49 7142 705-0
www.duerr.de
info@duerr.de